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RAPID HTA REPORT		
Request No.	Date of request	Applicant
294	06/06/2023	Sd Minimally Invasive Thoracic Surgery and Robotics - AOUP
Type of report		
New report		SI
Updating a previous report		
If updating, please state the reason:		

General technology data					
<b>Trade name</b>					
Signia Small Diameter Reload					
<b>Generic name</b>					
Disposable loader with digital sensor for tissue thickness measurement compatible with Signia™ Smart Stapling Platform and EndoGia™ suturers					
<b>Manufacturer name</b>					
Medtronic					
<b>Supplier name</b>					
Medtronic					
<b>RDM</b>	<b>REF</b>				
2031087, 2031088, 2031089	SIGSDL45CTVTVT, SIGSDS30CTVTV				
<b>Type</b>	<b>CE marking (date)</b>	<b>Risk class</b>	<b>FDA approval</b>		
CLASS I DM	13/09/2019	III	Yes, approval of 510(k) premarket notification in September 2019		
<b>CND</b>					
H020301050101 - SINGLE-PATIENT ARTICULATED LINEAR SUTURERS WITH THORACOSCOPY BLADE					
<b>Clinical problem and rationale for the request</b>					
Need for the use of smaller stem diameters for use in lung cancer surgery, conducted with a view to functional sparing and therefore segmental resections with smaller, thinner vessels. The smaller stem and anvil sizes as well as the two different operating stem lengths are necessary in robotic pulmonary lobectomy and robotic pulmonary segmentectomy procedures due to an advantage in terms of easy access, manoeuvrability and consequent safety during resection of small arterial and venous vascular structures.					
<b>Indications for use</b>					
Signia™ Small Diameter Reload white and grey refills find application in abdominal, gynaecological, paediatric and thoracic surgery for resection and transection of blood vessels (grey refills) and thin tissue and blood vessels (white refills).					
<b>Target patient</b>					
Oncology patients with lung cancer to undergo thoracic surgery.					
<b>Main competitors</b>					

**General technology data**

According to the requesting clinician, the comparator is suturers with larger diameter vascular loaders.

**Technological details**

**Description**

Signia™ Small Diameter Reload- Disposable loader with digital sensor for tissue thickness measurement, articulating 30 and 45 mm; length op. stem LONG 24 cm; SHORT 15 cm; diameter 8 mm. Fixed 6.3 mm anvil with gold curved dissecting tip. Two double rows of staggered stitches for tissue and section blade. Open stitch size 2.5-2.5 mm White for vascular tissue or 2.0-2.0 mm Grey for thin vascular tissue. Compatible with Signia™ Smart Stapler-EndoGia™Ultra [1].

**Elements of Innovation**

The Signia™ Small Diameter Reload (30 and 45 mm) has specific features such as: 1) it is the only refill to have a fixed anvil with a 6 mm wide tip; 2) an 8 mm diameter operating stem and an articulation of up to 50 degrees; 3) availability with these dimensions of 6 mm anvil width and 8 mm stem diameter and an articulation of 50 degrees, in both the 30 mm and 45 mm operating lengths (this is to allow an approach to both single and branched segmental vascular structures) 4) be present in the 30-mm working size with both 1-mm and 0.75-mm closed point height to be able to approach situations with segmental vessel wall thicknesses of even less than 1 mm.

These features, according to the requesting clinician, allow 75% better minimally invasive access than the Medtronic 12 mm refill. Sutures in use can be dangerous when used on small vessels due to traumatism that can lead to vessel rupture because they must be placed in minimal passages between isolated anatomical portions to perform minimal but oncologically safe resections such as segmentectomies.

The Signia™ Small Diameter Reload refills are compatible with both the Endogia Ultra manual handpiece and the Signia electronic handpiece, so in procedures where these refills are chosen, there is no need to use a specific handpiece than the one used in the lung and bronchial parenchyma section passages.

**Clinical and economic evidence**

**Clinical studies**

The Pubmed search with the search key "narrow-profile vascular stapler" and "Signa Small" revealed 13 studies, of which only one was relevant to the device being requested. This was a case report on a 70-year-old patient suffering from lung cancer and undergoing thoracotomy surgery with the da Vinci Robot and the use of the Signa Small loader [2].

**Ongoing clinical trials**

No study found on Clinicaltrial.gov on DM Signia Small Reload.

**Guidelines**

No Guideline on the DM required

**Cost-effectiveness analysis**

No cost effectiveness analysis detected

**HTA Report**

No HTA reports detected

**Expected benefits**

Ease of access, manoeuvrability and consequent safety during resection of arterial and small venous vascular structures during robotic pulmonary lobectomy and robotic pulmonary segmentectomy procedures.

**Price and cost of therapy per patient**

Product (Manufacturer)	Unit price (euro)	Therapy cost per patient (euro)	Annual requirements (No. pieces)
---------------------------	----------------------	------------------------------------	-------------------------------------

**Price and cost of therapy per patient**

Signia™ Small Diameter Reload (30 and 45 mm) (Medtronic)	540	1.080-1.620	35-40*
--	-----	-------------	--------

\*The need is limited to a small subset of patients considering a total of about 500 procedures per year.

**Price and cost of therapy per patient with existing treatment alternatives**

Product (Manufacturer)	Unit price (euro)	Therapy cost per patient (euro)
Ethicon Echelon Flex WHITE VASCULAR REFILL 35MM ALT.CLOSED STITCH 1.0MM FOR ECHELON POWERED SUTURING MACHINE REF.VASECR35 (Ethicon)	179,9 - 274,0	359,8-539,7*

\*considering the use of 2 or 3 refills per intervention.

**Economic and organisational impact**

The use of this device entails an increase in expenditure, which is, however, negligible given the small quantity required.

**Reimbursability information**

Main diagnosis (ICD9-CM code)	Intervention (ICD9-CM Code )	DRG code	Tariff (euro)
162.9	32.4	75	8.999

**Assessment of innovativeness (according to Regional Resolution N° 737/2022, [link](#) and N° 1244/2022 [link](#) )**

Innovative device (Y/N)	N
If yes, please indicate which Criteria 1, 2 and 3 are met:	

**Summary data**

Request number	Date of request	Applicant
294	06/06/2023	Sd Minimally Invasive Thoracic Surgery and Robotics - AOUP
<b>Technology</b>		
Signia Small Diameter Reload		
<b>Conclusions</b>		

### Summary data

Surgery is moving towards a direction of mini-invasiveness for which instruments of minimal size are required, especially for robotic surgery. Increasingly, lung resection surgeries are aimed at minimal resections; segmentectomies meet these requirements if supported by proportionally thinner sutures for intersegmental vessels and thus smaller than lobar vessels (lobectomies).

The Signia™ Small Diameter Reload suture refills fit into this context allowing, for the reasons stated above, better accessibility, manoeuvrability and consequent safety during the resection phase of small arterial and venous vascular structures, compared to the sutures already in use (Medtronic's 30 mm Tri Staple refills and Ethicon's 35 mm powered vascular suture, determines Estar 719/2021).

Critical issues are the unavailability of clinical evidence, apart from the case report [2], regarding the use of Signia™ Small Diameter Reload and the price of this device being higher than that of the comparator. Despite this, given the potential benefits of Signia™ Small and the small annual requirement, the CO expresses a favourable opinion.

### Date of report

04/10/2023

### Author(s) of the report

Sabrina Trippoli

### Referring pharmacist

Sunday Mamone

## BIBLIOGRAPHY

3. Signia™ Small Diameter Reload. Available at: <https://www.medtronic.com/covidien/it-it/products/surgical-stapling/signia-small-diameter-reload.html#>. Last accessed 04/10/2023
4. Shidei H, Mitsuboshi S, Akihiro K, Aoshima H, Isaka T, Kanzaki M. Robot- assisted thoracoscopic surgery using a narrow-profile vascular stapler through an 8-mm robotic port. J Surg Case Rep. 2021 Apr 24;2021(4):rjab157. doi:10.1093/jscr/rjab157.

A copy of the document can be downloaded from <http://www.regione.toscana.it/-/prodotti-hta>.

Reporting by the Operations Centre, Regional Decree No. 17610 of 7 September 2022.

For further information write to [centro.operativo.htart@regione.toscana.it](mailto:centro.operativo.htart@regione.toscana.it)