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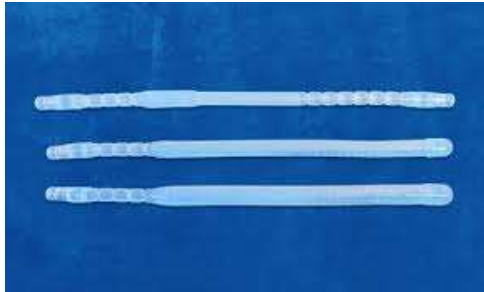
Domenica Mamone

BIBLIOGRAFIA

1. Shindel AW, Bullock TL, Brandes S. Urologist practice patterns in the management of Peyronie's disease: a nationwide survey. *J Sex Med* 2008; 5: 954—64.
2. Christopher JS, Chelsea M, Ridwan S. Peyronie's disease: the epidemiology, aetiology and clinical evaluation of deformity. *BJU Int* 2005; 95: 729—32.
3. Swarzer U, Sommer F, Klotz T, Braun M, Reifenrath B et al. The prevalence of Peyronie's disease: results of a large survey. *BJU Int* 2001; 88: 727—30.
4. Usta MF, Bivalacqua TJ, Tokatli Z, Rivera F, Gulkesen KH et al. Stratification of penile vascular pathologies in patients with Peyronie's disease and in man with erectile dysfunction according to age: a comparative study. *J Urol* 2004; 172: 259—62.
5. Vardi Y, Levine LA, Chen J, Hatzimouratidis K, Sohn M. Is there a place for conservative treatment in Peyronie's disease? *J Sex Med.* 2009 Apr;6(4):903-909. doi: 10.1111/j.1743-6109.2009.01214.x. PMID: 19338644.
6. Zucchi A, Silvani M, Pecoraro S. Corporoplasty with small soft axial prostheses (VIRILIS [®]) and bovine pericardial graft (HYDRIX[®]) in Peyronie's disease. *Asian J Androl.* 2013 Mar;15(2):275-9. doi: 10.1038/aja.2012.156. Epub 2013 Jan 28. PMID: 23353721; PMCID: PMC3739159.
7. Austoni E, Colombo F, Romanò AL, Guarneri A, Kartalas Goumas I, Cazzaniga A. Soft prosthesis implant and relaxing albugineal incision with saphenous grafting for surgical therapy of Peyronie's disease: a 5-year experience and long-term follow-up on 145 operated patients. *Eur Urol.* 2005 Feb;47(2):223-9; discussion 229. doi: 10.1016/j.eururo.2004.10.004. PMID: 15661418.
8. Zucchi A, Silvani M, Pecoraro S. Corporoplasty with small soft axial prostheses (VIRILIS [®]) and bovine pericardial graft (HYDRIX[®]) in Peyronie's disease. *Asian J Androl.* 2013 Mar;15(2):275-9. doi: 10.1038/aja.2012.156. Epub 2013 Jan 28. PMID: 23353721; PMCID: PMC3739159.
9. Spinozzi L, Droghetti M, Palmisano F, Piazza P, Fontanella L, Gentile G, Franceschelli A, Colombo F. Use of Soft Penile Prosthesis in Grafting Surgery for Peyronie's Disease and Mild Erectile Dysfunction: Still an Option? *Arch Esp Urol.* 2024 Apr;77(3):270-277. doi: 10.56434/j.arch.esp.urol.20247703.36. PMID: 38715168.
10. Silvani M, Pecoraro S, Zucchi A. Corporoplasty for induratio penis plastica with soft axial tutors, single relaxing albugineal incision and safenous grafting. A 3-year follow up. *Arch Ital Urol Androl.* 2012 Jun;84(2):84-8. PMID: 22908777.
11. Grasso M, Lania C, Fortuna F, Blanco S, Piacentini I. Preservation of cavernosal erectile function after soft penile prosthesis implant in Peyronie's disease: long-term followup. *Adv Urol.* 2008;2008:646052. doi: 10.1155/2008/646052. Epub 2008 Dec 2. PMID: 19081839; PMCID: PMC2593409.

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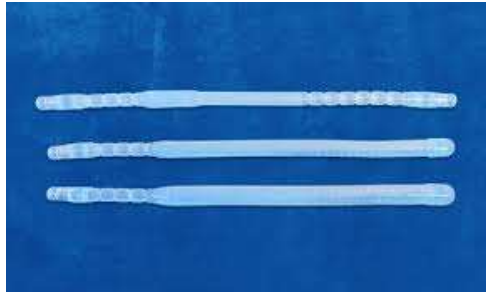
RAPID HTA REPORT		
No. of the request	Date of request	Applicant
328	08/03/2024	AOUP University Urology
Type of report		
New report		Yes
Update of a previous report		
If update, state the reason:		

General technology data					
Trade name					
Zephyr ZSI 100 CF					
Generic name					
HYDROPHILIC SOFT PENILE PROSTHESIS					
Manufacturer name					
ZSI -Zephyr Surgical Implants					
Supplier name					
ZSI -Zephyr Surgical Implants					
RDM	REF				
1466672	ZSI 100 D9 CF				
Type	CE mark (date)	Risk class	FDA approval		
1	03/03/2016	2b	No		
CND					
P080202 - Penile prostheses.					
Clinical problem and rationale for the request					
<p>Treatment of patients with La Peyronie's disease undergoing reconstructive andrological surgery.</p> <p>La Peyronie's disease is a condition whose etiology is, to date, still unknown. It is characterized by abnormal scarring of the tunica albuginea with loss of elasticity of the tunica albuginea and consequent penile deformities. The disease is always associated with loss of elasticity and shortening of the penis, which negatively affects the quality of (sexual) life of patients with serious relational and psychological consequences [1-6].</p> <p>The incidence of this disease is 3-10 % among the male population and can be commonly found in men between 40 and 70 years of age, although it can also affect individuals under 30 years of age [2-3]. In addition, Peyronie's disease is commonly associated with erectile dysfunction (20-54%).</p> <p>Surgical treatment must be individualized by aiming not only to support penile erectile function but also to restore the pre-existing anatomical and morphological conditions of the male genitalia. The technique of corporoplasty proposed by Austoni et al. [7] a few years ago is based on geometric principles and involves the prior placement of soft silicone penile prostheses within the corpora cavernosa; these silicone "braces" in fact allow axial extension of the penis making it easier to identify the curvature and the point of greatest traction, thus facilitating the operator during corporoplasty surgery [7-11].</p>					
Indications for use					
<p>A penile prosthesis is a device that allows patients to achieve and maintain an erection for:</p> <ul style="list-style-type: none"> -treating erectile dysfunction; -maintaining space in the corpora cavernosa; -Enabling intermittent self-catheterization; 					

General technology data	
General technology data	-Keep the penile sheath in place; -treating Peyronie's disease.
Target patient	Patients with La Peyronie's disease and erectile deficiency.
Main competitors	Hydraulic tricomponent prosthesis implantation; this is a costly procedure (see below). Until a few years ago, a soft prosthesis, Virilis (Giant Medical company), superimposed on ZSI 100 CF, was available; now that prosthesis is off the market.
Technology details	
Description	The ZSI 100 CF penile prosthesis is a device that enables patients to achieve an erection. This implant consists of two soft cylinders (endocavernous prosthesis) inserted into the penis. The prosthesis is made mainly of silicone. The prosthesis is available in three different diameters to fit the anatomy of patients' corpora cavernosa: 9 mm, 11 mm and 13 mm. Adjustable from 25 cm to 12 cm.
Elements of innovation	<p>The "soft brace" is not a true prosthesis in the strict sense of the word, but precisely a brace, since it does not replace the erection (as in the case of hydraulic prostheses) but, by using the spared cavernous tissue, supports the patient's residual spontaneous erection giving a natural effect to the sexual act compared to other prosthetic implants. In addition, the brace, by constantly stretching the corpora cavernosa, prevents scar retraction, after grafting of a biocompatible patch, or recurrence of fibrosis plaques.</p> <p>The ZSI 100 CF silicone soft prosthesis proposed by Zephyr has the features necessary for implantation during reconstructive surgery. The Zephyr ZSI size 9 CH, which is useful precisely in this type of surgery where it is necessary to place small braces in order to leave as much space as possible for the residual cavernous tissue within the corpora cavernosa. In this way the brace fills the corpus cavernosum about 70% of the space leaving 30% for spontaneous cavernous erectile activity. The brace has good axial rigidity that allows us to extend the penis longitudinally during surgery, emphasizing the curvature due to the disease but also extending the corpus cavernosum after the fibrosis plate is incised and the traction released. In this way, the end result is aesthetically very good as the penis has a natural stance while regaining a good morphological and functional appearance. The length of the prosthesis can be easily adjusted by sectioning it along a graduated scale as needed.</p> <p>The significantly reduced costs, compared with standard hydraulic prostheses, make it a valuable compromise in many cases of andrologic reconstructive surgery.</p>
Clinical and economic evidence	
Clinical studies	<p>The literature search selected 5 articles none of which are relevant to the evaluation of the device being evaluated.</p> <p>MEDLINE search PubMed version (site: www.pubmed.org) conducted on 09/20/2024; keywords "zephyr soft."</p>
Ongoing clinical trials	Clinicaltrials.gov reports only two studies that are still in the recruitment phase, but these do not involve La Peyronie's disease, but rather transgender intervention situations.
Guidelines	None

Cost-effectiveness analysis
None
HTA Report
None
Expected benefits
Functional improvement and reduction of expenditure per patient.

Price and cost of therapy per patient			
Product (Manufacturer)	Unit price (euro)	Cost of therapy per patient (euro)	Annual requirement (No. pieces)
Zephyr ZSI 100 CF (Zephyr Surgical Implants)	1.976	ditto	4

Price and cost of therapy per patient with treatment alternatives already in use		
Product (Manufacturer)	Unit price (euro)	Cost of therapy per patient (euro)
Hydraulic Tricomponent Prosthesis (Peniena Tricompon X Imp Infrapubic 20cm Titan Touch + Cloverleaf 125cc Reservoir for Peniena Tricomponent Titan Touch Prosthesis)		6,400 prosthesis + 1,300 tank

Economic and organizational impact
In line with the above.

Reimbursability information			
Principal diagnosis (ICD9-CM code).	Intervention (ICD9-CM Code)	DRG Code	Rate (euro)
64.95	607.85 + 607.84	341	3,856.00 ordinary hospitalization > 1 day

Evaluation of innovativeness (according to Regional Resolution No. 737/2022, link and No. 1244/2022 link)	
Innovative device (Y/N)	No
If yes, please indicate which Criteria 1, 2 and 3 are met:	

Summary data		
Request number	Date of request	Applicant
328	08/03/2024	AOUP University Urology
Technology		
Zephyr ZSI 100 CF soft penile implant		

Summary data

Conclusions

The soft penile prostheses having a very small diameter (9-10 mm) can be implanted through a single cavernous dilatation with sparing of peripheral erectile tissue, thus preserving the complementary residual erection, an erection that, combined with the axial rigidity of the prosthetic cylinders themselves, ensures good vaginal penetration and functionally satisfactory results.

Zephyr ZSI 100 CF has no studies to support it, while trials [7-11] are available on the Virilis soft prosthesis (completely superimposable to ZSI 100 CF) that is now out of production. The CO favors the purchase of Zephyr ZSI 100 CF both because of the economic advantage in the management of the patient thus treated and in view of the small number (N=4) of case histories required.

Date of report writing

August 2024

Author(s) of the report

Andrea Messori

Referring pharmacist

Domenica Mamone

BIBLIOGRAPHY

1. Shindel AW, Bullock TL, Brandes S. Urologist practice patterns in the management of Peyronie's disease: a nationwide survey. *J Sex Med* 2008; 5: 954-64.
2. Christopher JS, Chelsea M, Ridwan S. Peyronie's disease: the epidemiology, aetiology and clinical evaluation of deformity. *BJU Int* 2005; 95: 729-32.
3. Swarzer U, Sommer F, Klotz T, Braun M, Reifenrath B et al. The prevalence of Peyronie's disease: results of a large survey. *BJU Int* 2001; 88: 727-30.
4. Usta MF, Bivalacqua TJ, Tokatli Z, Rivera F, Gulkesen KH et al. Stratification of penile vascular pathologies in patients with Peyronie's disease and in men with erectile dysfunction according to age: a comparative study. *J Urol* 2004; 172: 259-62.
5. Vardi Y, Levine LA, Chen J, Hatzimouratidis K, Sohn M. Is there a place for conservative treatment in Peyronie's disease? *J Sex Med.* 2009 Apr;6(4):903-909. doi: 10.1111/j.1743-6109.2009.01214.x. PMID: 19338644.
6. Zucchi A, Silvani M, Pecoraro S. Corporoplasty with small soft axial prostheses (VIRILIS I®) and bovine pericardial graft (HYDRIX®) in Peyronie's disease. *Asian J Androl.* 2013 Mar;15(2):275-9. doi: 10.1038/aja.2012.156. Epub 2013 Jan 28. PMID: 23353721; PMCID: PMC3739159.
7. Austoni E, Colombo F, Romanò AL, Guarneri A, Kartalas Goumas I, Cazzaniga A. Soft prosthesis implant and relaxing albugineal incision with saphenous grafting for surgical therapy of Peyronie's disease: a 5-year experience and long-term follow-up on 145 operated patients. *Eur Urol.* 2005 Feb;47(2):223-9; discussion 229. doi: 10.1016/j.eururo.2004.10.004. PMID: 15661418.
8. Zucchi A, Silvani M, Pecoraro S. Corporoplasty with small soft axial prostheses (VIRILIS I®) and bovine pericardial graft (HYDRIX®) in Peyronie's disease. *Asian J Androl.* 2013 Mar;15(2):275-9. doi: 10.1038/aja.2012.156. Epub 2013 Jan 28. PMID: 23353721; PMCID: PMC3739159.
9. Spinozzi L, Droghetti M, Palmisano F, Piazza P, Fontanella L, Gentile G, Franceschelli A, Colombo F. Use of Soft Penile Prosthesis in Grafting Surgery for Peyronie's Disease and Mild Erectile Dysfunction: Still an Option? *Arch Esp Urol.* 2024 Apr;77(3):270-277. doi: 10.56434/j.arch.esp.urol.20247703.36. PMID: 38715168.
10. Silvani M, Pecoraro S, Zucchi A. Corporoplasty for induratio penis plastica with soft axial tutors, single relaxing albugineal incision and safenous grafting. A 3-year follow-up. *Arch Ital Urol Androl.* 2012 Jun;84(2):84-8. PMID: 22908777.
11. Grasso M, Lania C, Fortuna F, Blanco S, Piacentini I. Preservation of cavernosal erectile function after soft penile prosthesis implant in Peyronie's disease: long-term followup. *Adv Urol.* 2008;2008:646052. doi: 10.1155/2008/646052. Epub 2008 Dec 2. PMID: 19081839; PMCID: PMC2593409.

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Centro operativo

A copy of the document can be downloaded from <http://www.regione.toscana.it/-/prodotti-hta>.

Drafting of the report by the Operations Center, Regional Decree No.17610 of September 7, 2022.

For more information email centro.operativo.htart@regione.toscana.it